







Specimen Collection Containers

	<p>Lavender (EDTA)</p>	<p>Recommendation to fill between the 250 and 500 lines on the tube.</p>
	<p>Na Citrate (Blue top)</p>	<p>Call Hematology 22415 to have tubes sent up. Collect 0.9 ml of blood and fill to the line for 1.0 ml total. Return all unused tubes to the lab.</p>
	<p>Green (Lithium Heparin) Plasma</p>	
	<p>Yellow Top Gel Serum</p>	<p>Used for Digoxin, Insulin, therapeutic drug monitoring and growth hormone.</p>
	<p>Plain Red -No Gel- Serum</p>	<p>Use for Caffeine testing</p>
	<p>Green (Na Heparin)</p>	<p>Use only for Chromosomes</p>