


Blue Top Tubes (3.2% Sodium Citrate)

Blue top tubes (3.2% Na Citrate) should be filled as below. Tubes should be mixed immediately after drawing by gently inverting 180° and back, 3-4 times.

Manufactured by BD

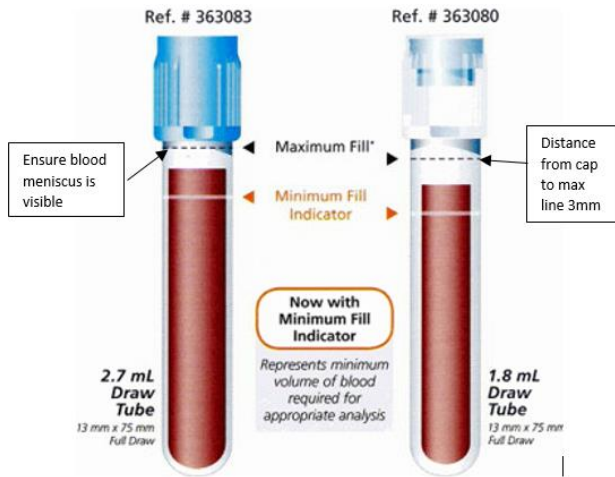

Manufactured by Greiner



Sodium citrate microtainers should only be used for venous or arterial blood. The tube should be filled with 0.9 ml blood drawn with a syringe. The tube should be filled to the 1.0 ml line. Tubes should be mixed after filling. Ensure that the blood actually moves from end to end in the tube.



Effective 8/29/23